



Interpath Laboratory, Inc. Test File Update

As an Interpath customer who receives electronic results or sends electronic orders you may need to be notified when we update our Service Manual. Although we try to keep these changes to a minimum, laboratory medicine is an evolving industry requiring changes to our technology from time to time. Depending on the requirements of your EMR or Hospital Information System you may be required to make similar changes to your system in order to correctly process inbound electronic results and create outbound electronic orders.

If you are uncertain that you are required to update your system we recommend that you contact your vendor for more information. As your laboratory service provider we are available to participate in the discussion with your vendor so that you clearly understand the impact of these changes.

Included in this email:

- This cover letter with a summary of the changes
- Microsoft Word® Document with the detail of these changes to our Service Manual
- Interpath Master Order/Result Compendium

Additional information including our most recent Service Manual and additional contact information can be found at www.interpathlab.com

Effective Date: March 12, 2020

Order Code	Test Name	NC	CC	CPT	SRC	RRC	NT	DT	AOE
		Name Change	Component Change	CPT Change	Specimen Requirements Change	Reference Range Change	New Test	Discontinued Test	Ask on Order Entry Questions
92200	2019 Novel Coronavirus SARS-CoV-2 by PCR						◆		

92200 2019 Novel Coronavirus SARS-CoV-2 by PCR
NT

Specimen:	
Collect:	One Nasopharyngeal Swab in Viral Transport Media One Throat Swab in Viral Transport Media
Submit:	Swab in Viral Transport Media. Submit Frozen.
Special Handling:	State Source and Patient Address. Collect: Nasopharyngeal swab AND oropharyngeal swab. Place both swabs in one collection tube. Transport: Place both swabs in a single viral transport media, place each specimen in an individually sealed bag. Transport frozen.
Rejection Criteria:	Specimens not in viral transport media. Containers that are leaking or otherwise compromised. Calcium alginate or cotton swabs. Swabs with wooden shafts.
Stability:	Ambient: Unacceptable; Refrigerated: 4 Day(s); Frozen: 1 Month(s); Incubated: Unacceptable
Methodology:	Qualitative Polymerase Chain Reaction (PCR)
Performed:	Sun-Sat
Reported:	3-5 Day(s)
CPT Codes:	U0002
Interpretive Data:	Please see report for interpretive data.
Components:	92201 - SARS-CoV-2 SOURCE 92202 - SARS-CoV-2 by PCR

New test available. See attached document for further COVID-19 collection information.